Bone Cement Cemento chirurgico

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Product Description

LIMA-CMT 1, LIMA-CMT 3 and LIMA-CMT G are orthopaedic acrylic radiopaque sterile cements, manufactured in compliance with ISO 5833: 2002.

Given their self-curing specification, they allow an immediate and stable fixation of surgical implants into the bone during orthopaedic surgery. According to the surgeon's requirements, he will be able to select the most suitable cement within our range.

These cements are made of two sterile components: powder polymer and liquid monomer. These two components are packaged in two sterile packs.

Each pack contains a liquid vial and a powder unit, both included inside a blister pack closed by a peelable tyvek, the whole being packaged in a box. The packages are sterilized by ethylene oxide.

The components are for single use only.

Do not re-sterilize the components. The sterilization is guaranteed only if the package is undamaged.

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Liquid
Liquido

Liquid Component

The liquid is composed of methylmethacrylate, butylmethacrylate (co-monomer), N:N dimethyl-p-toluidine and hydroquinone.

The liquid is flammable, its odour is very distinctive. It is sterilized by ultra-filtration.

Powder Component

The major component is polymethylmethacrylate.

The benzoyl peroxide, which initiates the polymerisation, is included in polymethylmethacrylate.

In order to get the required reaction and viscosity, various polymethylmethacrylates are mixed in adequate proportions. Barium sulphate is included up to 10% by unit and complies with the European Pharmacopoeia.

The bone cement powder is sterilized by beta radiation (25 kGy).

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Materials: Plugs

Cement plugs are manufactured in UHMWPE and have a radio opaque reference in AISI 316L.

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Indications

The cement plugs are used to tap the femoral canal before cement injection and avoid cement flow in the diaphysis (hip arthroplasty).
They can be used with any Lima cemented stem (Note: the Friendly stem has got its own unique set of plugs).

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Surgical Technique

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[1]

The instrument set contains the inserters and the trial plugs.

The trial plugs are available in the sizes: 8.5; 10; 11.25; 12.5; 13.75; 15; 16.25; 17.5; 18.75; 20 mm.

To verify the size of the definitive plug you have to screw it to the inserter and fit it in the femoral canal (Fig. 1). Proceed increasing the size of the trial plugs to check the diameter of the femoral canal.

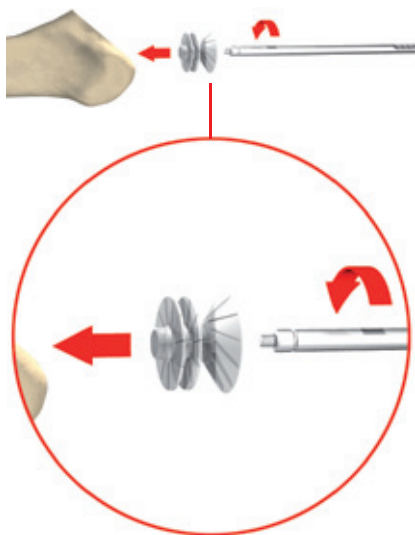
If trial plugs from 8.5 to 11.25 fit in the femoral canal then, the suggested size is 1.

If trial plugs from 12.5 to 17.5 fit in the femoral canal then, the suggested size is 2.

If trial plugs from 18.75 or more fit in the femoral canal then, the suggested size is 3.

By screwing the definitive plug to the inserter is possible to position the plug in the femoral canal (Fig. 2).

It is advisable to position the cement plug at the distance of 1 cm distally from the tip of the stem.

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
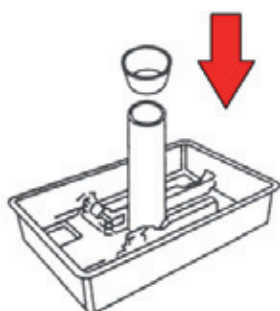
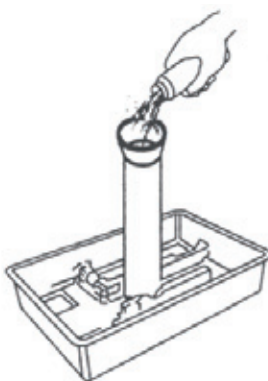
Pour powder
Versare polvere

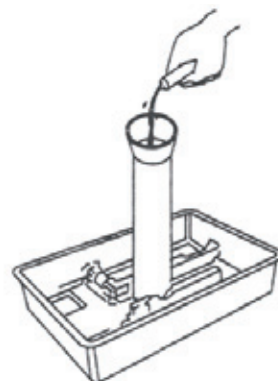Pour liquid
Versare liquido

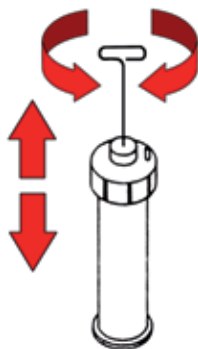
Warning! Screw the ring nut tightly before mixing

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After putting the elements together, they can be mixed using the "operating room vacuum system" and normally in air as well.

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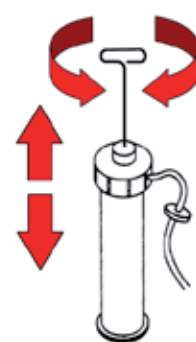
Our mixing method considerably increases the technical characteristics of the cement. Removing all air bubbles and the emission of free monomer gas through a constant aspiration.

After the mixing phase, the cement may be applied by injection or manually.

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


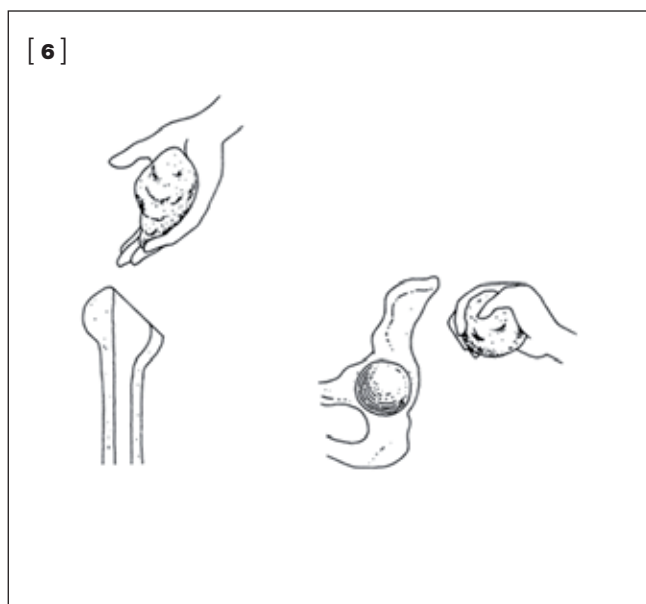
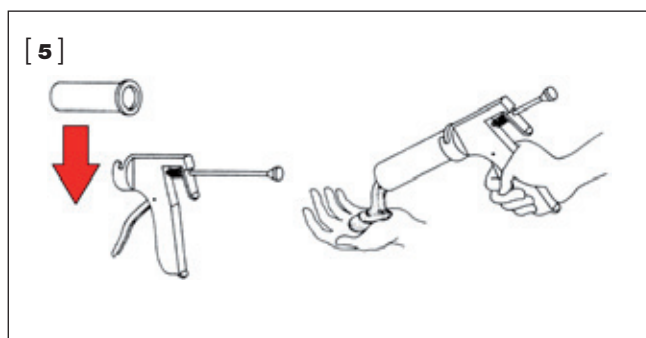
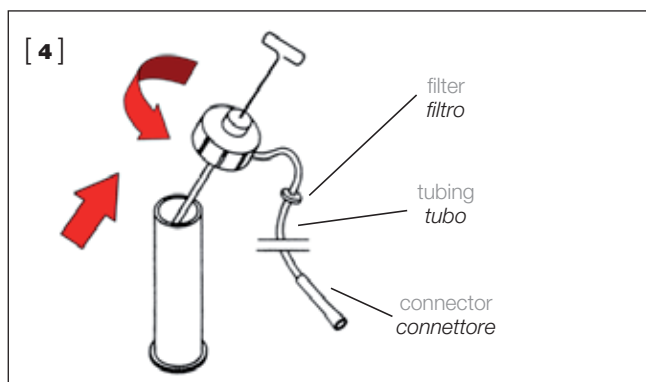
LIMA-CMT 1, LIMA-CMT 3 AND LIMA-CMT G CEMENT:
MANUAL APPLICATION

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LIMA-CMT 3 CEMENT:
APPLICATION BY INJECTION

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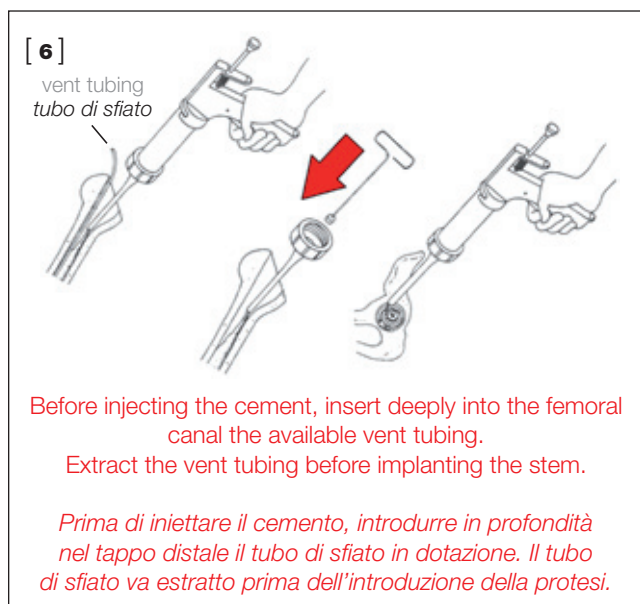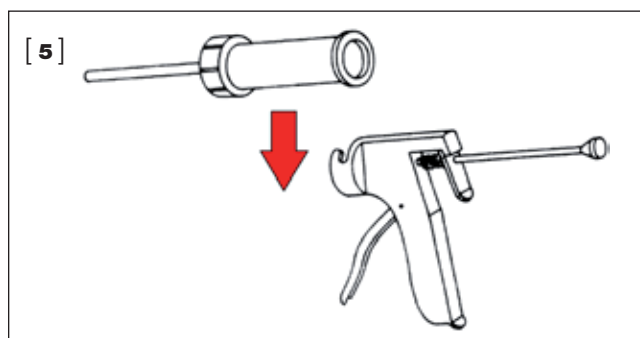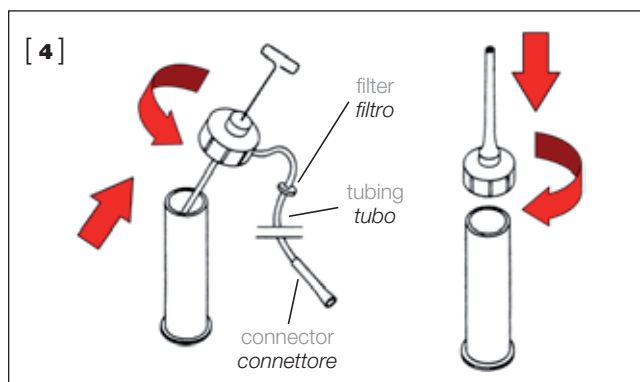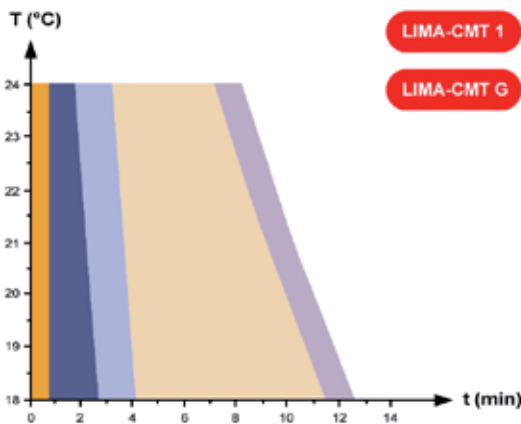
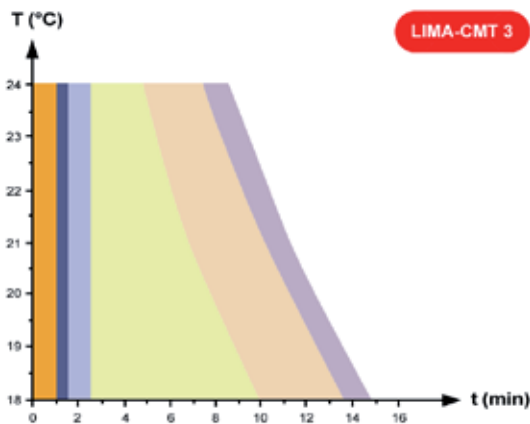
Pressurization Kit: LIMA-CMT 3

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Note. To be used in combination with Lima's Cartridge Complete Kit cod. 9096.30.100.

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[1]

Irrigate and dry the canal with the available tampon

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
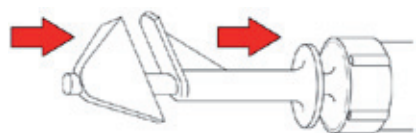
[3]

Clean the nozzle then fit the proximal centralizer presser and the seal onto the nozzle

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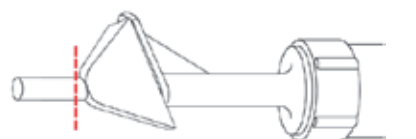[4]

Break the syringe nozzle under the seal

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[5]

Insert the syringe, press the seal against the resection surface and inject more cement

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This ensures that the cement is highly pressurized. Now remove the entire device and remove thoroughly any exceeding cement.

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The third Generation Cementing Technique, thanks to the combined use of low viscosity cement, distal plugs, special pressurization kits and enhanced design femoral stems, reduces cement porosity, increases uniformity of cement and quality of fixation.

This technique needs careful irrigation and drying with tampon to prevent blood interposition.

The low viscosity cement is injected filling the femoral canal from the distal part to the proximal part with a long nozzle cement gun, after having introduced deeply a vent tubing into the distal plug.

To complete the filling the vent tubing and the nozzle are extracted. A seal is inserted onto the nozzle which is shortened by breaking.

The seal allows the pressurization which is generated by means of the gun.

The experience confirms that the use of an hermetic proximal seal during cement pressurization has resulted in a marked improvement in clinical results in cemented femoral components at medium-term follow-up periods.

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[illegible]

This surgical technique is provided for purely illustrative reasons.

The surgeon must always proceed on the basis of his personal experience and considering the conditions of each individual patient.

For further information about our products, please visit our web site www.limacorporate.com

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Lima Orthopaedics Australia Pty Ltd
Lima Orthopaedics New Zealand Ltd
Lima Orthopaedics UK Limited